# LUXAMED®

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**LUXAMED® Tuning Forks**

**Diapasons LUXAMED®**

**Diapason LUXAMED®**

**Diapasones LUXAMED®**

**Luxamed Kamertony®**

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Deutsch .....	2
English .....	6
Français .....	10
Italiano .....	14
Español .....	18
Polski.....	22
Portuguese .....	26

# LUXAMED®

Hersteller: LUXAMED GmbH & Co. KG \*  
Daniel-Weil-Str.3 \* 89143 Blaubeuren \* Germany  
Telefon: +49(0)7344 92905 0 \* Fax: +49(0)7344 92905 10  
E-Mail: info@luxamed.de \* Internet: www.luxamed.de

**- DEUTSCH -**

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**- ENGLISH -**

To ensure proper use of this LUXAMED® product, please read this user manual carefully and retain for future reference.

## **LUXAMED® Tuning Forks**

### **Intended use of instrument:**

The LUXAMED® Tuning forks will be used solely as an instrument for testing the nerve sensitivity and the sound line on healthy skin.

These instruments are only to be used by authorized, trained personnel. Keep out of the reach of children.

Use the device in accordance with its designated use and for its intended purpose only.

### **Contraindication:**

none

## **Warning:**

Please make sure that, prior to any use; the plastic stands are fixed correctly at the tuning fork. NEVER use without plastic stand.

All serious incidents that have occurred with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

## **Application:**

The forks are placed on the body zones which are to be tested for functionality. Care must be taken in a cautious application. By the vibration of the tuning forks, a diagnosis can be made.

## **Cleaning:**

Always clean the tuning fork right after use. The tuning fork can be cleaned with a damp cloth (use ph-neutral or slightly alkaline cleaning materials).

## **Disinfection:**

Disinfection or disinfecting cleaning of the outer surfaces of the tuning fork simply by wipe disinfection. For this, isopropanol can be used.

## **Sterilization:**

No sterilization necessary, not approved for sterilization.

## **Disposal:**

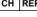
This product and its accessories can be disposed of along by with metal waste.

## Storage, transport and operating conditions:

Ambient temperature between -10° and +40°

Rel. humidity between 10% and 90%

## Symbol key

	Manufacturer
	Date of manufacture
	Attention to the user manual
	Caution
	In Conformity with EU guidelines
	It is a medical device
	Conforms with UK regulations
	Authorised agent for medical products in Switzerland: Inpera GmbH Thunstrasse 64 CH-3110 Münsingen, Switzerland

## **Warranty Conditions:**

We give 2 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge.

After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty. No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

LUXAMED GmbH & Co. KG  
Daniel-Weil-Straße 3  
89143 Blaubeuren  
GERMANY  
Tel.: + 49 (0) 7344 92905-0

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Representante autorizado suíço para dispositivos médicos: Inspera GmbH Thunstrasse 64 CH-3110 Münsingen